



ALCOHOL SWAB (STANDARD)

- 70% Isopropyl Alcohol Swab (with non woven swab)
- Fabric Size: folded 30mm x 30mm
- Thickness: 50 gsm
- 2 swabs per strip
- 50 strips per inner box (100/box)
- 100 inner boxes per outer carton
- Carton of 10,000 pcs

CHARACTERISTICS:

- Soft and sanitary
- Packaging is easy to tear and use
- Contains 0.5ml alcohol to maintain wetness of the pad

INTENDED USE:

The Alcohol Swabs are intended for preparation of the skin by disinfecting the injection site prior to injection or venipuncture. For professional and hospital use. For external use only.

STORAGE CONDITION:

The product shall be stored in dry, cool and clean location.

INSTRUCTION FOR USE:

- Take pad out of package
- Wipe injection site vigorously
- Discard after single use
- Do not use if package is opened or damaged

WARNINGS:

- For external use only.
- Keep away from fire or flame.
- Do not use with electro-cautery procedure
- Stop using if irritation or redness develop
- Keep out of reach of children

MDA REGISTRATION NO. : GA7890219-34888

